WALDHAUSEN

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W - Health & Care

INHALIERGERÄT

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Please read this user manual carefully, store it for future reference, make it available to other users and follow the instructions.

WARNING

- The device is only intended for private use at home, and not for commercial use.
- This device may be used by children aged 8 years and older and by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been supervised or instructed in the safe use of the device and understand the hazards involved.
- This device is not a toy.
- Cleaning and user maintenance must not be carried out by children unless they are supervised.
- If the power cord of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- The device must never be opened or repaired under any circumstances; otherwise, proper function is no longer guaranteed. Failure to comply will void the warranty and liability.

Dear Customer,

We are delighted that you have chosen a product from the W-Health & Care range. Please read the user manual carefully and keep it in a safe place for future reference.

Kind regards,
The Waldhausen team

CONTENTS

1. Package contents.....	3
2. Signs and symbols.....	4
3. Getting to know the device.....	5
4. Warning and safety information.....	5
5. Device description.....	7
6. Commissioning.....	8
7. Filter change.....	9
8. Care and disinfection.....	10
9. Spare parts.....	10
10. Technical specifications.....	11
11. Disposal.....	11


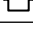
1. PACKAGE CONTENTS

Check the package to ensure that the outside of the cardboard packaging is intact and that all contents are present. Before use, make sure that there is no visible damage to the device and its accessories. Remove all packaging material. If in doubt, do not use the device and contact your trusted dealer.

- Inhaler
- Device cable
- Nebuliser
- Connection hose
- 5 Replacement filter
- Mouthpiece
- Storage bag
- User manual

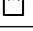2. SIGNS AND SYMBOLS

	WARNING Warning notice about risk of injury or dangers to your health.
	WARNING Safety notice on possible damage to the device/accessories.
	NOTE Reference to important information.
	The device has double protective insulation and therefore conforms to protection class 2
	Read the user manual.
	For indoor use only.
	Dispose of packaging in an environmentally friendly manner.
	Dispose of the device in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer
	CE MARK This product meets the requirements of the applicable European and national guidelines.
	The products verifiably meet the requirements of the technical regulations of the EAEU.

3. GETTING TO KNOW THE DEVICE

This W-Health & Care inhaler is an easy, practical means of treating your horse. Suitable for inhalation in the lower and upper respiratory tract and for deep inhalation.

4. WARNING AND SAFETY INFORMATION

Please read this user manual carefully. Failure to observe the following information can cause injury to humans or animals, or damage to property. Store this user manual in a safe place and make it available to other users. Hand this user manual over together with the device when you pass it on to a new owner.

WARNING

Keep the packaging material out of the reach of children due to the danger of suffocation.

WARNING

As with any electrical device, this inhaler should be used with care and caution in order to avoid the risk of an electric shock.

Product information

1. To avoid an electric shock:
 - Keep the device away from water.
 - Do not immerse the device or power cable in liquids.
 - Do not reach for any device that has fallen into water and disconnect the mains plug immediately.
2. Never operate the device if it is damaged (including the power cable), if it has been dropped or if it has been immersed in water. If in doubt, please contact your trusted specialist dealer.
3. The device must not be used if flammable gases, oxygen or aerosol spray products are being used.
4. Disconnect the device from the electrical outlet before cleaning or filling and after each use.
5. Do not use any other spare or add-on parts.

E **Precautions during use**

1. Connect the product to a suitable electrical outlet. Do not leave the device unattended during operation.
2. The device must not be operated if the cable or the plug is damaged, if it has been dropped or if it has been dropped into water in any way, or if it is not working properly.
3. If any anomalies occur, stop operation immediately and contact your trusted specialist dealer.
4. The product must always be disconnected the power supply after use.
5. Do not cover the air openings of the main unit and do not place it in a location where the air openings could be covered.

Precautions for storage

1. Do not store the device in places where it will be exposed to direct sunlight, high temperatures or humidity.
2. Store the device out of the reach of children.
3. Disconnect the mains plug whenever the device is not in use.

Precautions for cleaning

1. Do not immerse the device in water as this could damage the device.
2. Disconnect the device from its electrical outlet before cleaning.
3. Clean all necessary parts after each use (see Care and disinfection).

Repairs

WARNING

Repairs to electrical devices must only be carried out by specialised personnel. Improper repairs may result in considerable danger for the user. In the case of repairs, please get in touch with customer service or an authorised dealer.

5. DEVICE DESCRIPTION

6. COMMISSIONING

1. Assembling the inhaler

- Insert the hose into the hose connection of the compressor with a slight twist.
- Connect the other end to the nebuliser.
- Attach the nebuliser to the hose of the inhalation basket with the inhalation basket connection.

2. Filling the nebuliser

- Open the cover of the nebuliser.
- Pour an isotonic saline solution or the drug in the prescribed amount into the nebuliser. Do not overfill. The maximum recommended fill amount is 8 ml.
- If the required amount of medication is less than 2 ml, make up this amount with at least 4 ml of isotonic saline solution. Even viscous drugs may need to be diluted.

3. Closing the nebuliser
 - Close the cover of the nebuliser.
4. Connecting the accessories
 - Connect the compressor to the power supply. Insert the plug into a suitable socket.
 - Make sure that the ventilation slots remain free at all times.
5. Switching the device on A spray jet from the nebuliser indicates that the device is working properly. Put the inhalation basket on the horse and start the inhalation process.

7. FILTER CHANGE

Check the filter regularly (e.g. after every tenth use). Replace the filter if it is dirty or clogged, and after no more than 30 days. If the filter has become damp, it must be replaced immediately by a new one.

WARNING


Do not attempt to clean and reuse the filter. Use only original filters from the manufacturer, otherwise your device may break or effective inhalation cannot be guaranteed. Never use the nebuliser without a filter.

To replace the filter, follow these steps:

WARNING

First, switch the device off and disconnect the plug. Allow the device to cool.

1. Remove the filter cover with an upwards movement.
(If the filter remains in the device after removing the lid, remove the filter from the device using tweezers or similar.)
2. Replace the filter cover with the new filter.
3. Check that the filter cap is tightly in place.

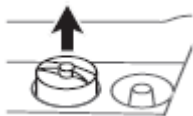8. CARE AND DISINFECTION

WARNING

Observe the following hygiene regulations to avoid health risks.

The nebuliser and the accessories are intended for multiple use. Please note that cleaning requirements differ according to the different application areas.

Preparation

Immediately after each treatment, all parts of the nebuliser and all accessories used must be cleaned of medication residues and impurities. To do this, remove the inhalation basket from the hose and nebuliser. To disassemble the nebuliser, turn the insert anticlockwise and pull it upwards. To reassemble the device later, perform these actions in reverse order.

Cleaning

Clean the inhalation basket, nebuliser and hose using distilled water.

For particularly intensive cleaning, place the individual parts in medical alcohol for about 30 minutes and then rinse them with hot water. Allow them to air dry or wipe them dry. Before reassembling the device, make sure that everything has dried well.

Wipe the compressor with a damp cloth. Make sure that no liquid gets into the ventilation slots.

Care of the connection hose: Switch the compressor on. Run the compressor until the air flowing through the hose has eliminated any moisture precipitation in the hose.

WARNING

Make sure that no water penetrates into the interior of the compressor. Do not place the device and accessories in a dishwasher. Do not touch the device with wet hands while it is plugged in. Do not allow the device to be splashed with water. The device may be operated only when it is completely dry. Do not spray liquids into the ventilation slots. Any liquid that penetrates into the the device may damage the electrical parts or other components of the nebuliser and affect the future operation of the device.

Disinfection

Carefully follow the steps listed below to disinfect your nebuliser and the accessories. It is recommended to disinfect the individual components after each use. There are two ways of disinfecting the nebuliser and the accessories: with vinegar or with boiling water.

Clean the nebuliser and the accessories as described in the “Cleaning” section. You can then continue with disinfection.

Disinfection with vinegar

- Use a vinegar solution consisting of 1/4 vinegar and 3/4 distilled water. Make sure that the volume is sufficient to completely immerse the parts such as the nebuliser, mask and mouthpiece in the solution.
- Leave the components in the vinegar solution for 30 minutes.
- Next, rinse the nebuliser and the individual components, such as the inhalation basket and connection hose, with water.
- Disinfection with boiling water:
- Place the nebuliser and connection hose in boiling water for about 10 minutes. Prevent the components from coming into contact with the hot bottom of the pot.
- Wipe the inhalation basket with boiling water.

9. SPARE PARTS

156822 Nebuliser, complete

156823 Connection hose

156808 Inhalation basket

156824 Filter with screw

10. TECHNICAL

Product name:	W-Health & Care Inhaler
Product model:	156820
Net weight:	1,3 kg (3,3 Lbs)
Power:	AC 230V.50 Hz
Power consumption:	180 VA
Drug capacity:	8 ml
Particle size	Approx. 75.74±0.02 % within 0.5 to 5 µm range
MMAD:	2.81±0,14 µm
Noise level:	Below 65dBA (DIN EN 13544-1)
Average nebulisation rate:	≥ 0,2 ml/min
Compressor pressure range:	30 – 50 Psi / 206 Kpa ~ 345 Kpa
Operating pressure range:	9.5 – 20 Psi / 65.5 Kpa – 137 Kpa
Litre flow range:	6-10 L/Min
Operating temperature range:	10°C to 40°C (50°F to 104°F)
Operating humidity range:	10 to 95 % RH
Storage temperature range:	-20°C to 70°C (-4°F to 158°F)
Storage humidity range:	10 to 95 % RH
Dimensions (L x W x H):	190 x 150 x 135 mm

11. DISPOSAL

WARNING

For environmental reasons, the device may not be disposed of in household waste at the end of its useful life, and must be disposed of at a suitable local collection or recycling point. Dispose of the device in accordance with the EC WEEE (Waste Electrical and Electronic Equipment) Directive. If you have any questions, please contact the municipal department responsible for waste disposal.

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WALDHAUSEN GmbH & Co. KG