

SIZE FINDER



HOOF BOOT QUALITY 
MADE IN SWITZERLAND



SG



SG-L

OUR BOOTS

Our best selling boot "SG" covers most hoof shapes and comes in six sizes. If your horse has a higher hoof heel and/or more pronounced bulbs, choose the "SG-L". This slightly longer boot comes in six sizes as well.

SG3 - SG8	= SG
SG3-X - SG8-X	= SG with spikes
SG3-L - SG8-L	= SG-L
SG3-LX - SG8-LX	= SG-L with spikes



WIDTH



LENGTH



HEIGHT



Both "SG" and "SG-L" are the perfect all-year boot. For snowy/icy seasons, check out our "SG-X" winter boot that comes with spikes.

SG3	SG3-X	106 - 111	to → 135	to → 59
SG3-L	SG3-LX	106 - 111	from 136 →	from 60 →
SG4	SG4-X	112 - 118	to → 145	to → 64
SG4-L	SG4-LX	112 - 118	from 146 →	from 65 →
SG5	SG5-X	119 - 125	to → 155	to → 69
SG5-L	SG5-LX	119 - 125	from 156 →	from 70 →
SG6	SG6-X	126 - 132	to → 165	to → 74
SG6-L	SG6-LX	126 - 132	from 166 →	from 75 →
SG7	SG7-X	133 - 140	to → 175	to → 79
SG7-L	SG7-LX	133 - 140	from 176 →	from 80 →
SG8	SG8-X	141 - 151	to → 185	to → 84
SG8-L	SG8-LX	141 - 151	from 186 →	from 85 →

All measurements in mm. Status: July 2021

SG-X



SWISS-GALOPPERS.COM



HOW TO MEASURE YOUR HORSES HOOVES

Caution: Pls. make sure to thoroughly measure in a 90° angle. The image below shows how it should not be done :-). A "wrong" angle can impact the measurements in an unbenefticial way.




When taking measures of the width, make sure you measure the widest part of the hoof.




When taking measures of the length, make sure you measure from the tip of the toe to the end of the bulbs.



When taking measures of the height, make sure you measure from the ground to the deepest part of the pastern.

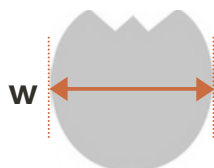

The width of the hoof determines the size of the boot (3-8).

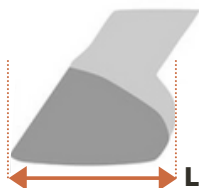

The length of the hoof determines whether you need a "SG" or a "SG-L".



A specific height will indicate a "SG-L" is needed.

WIDTH (in mm)

LENGTH (in mm)

HEIGHT (in mm)

Front left
Front right
Hind left
Hind right

.
.
.
.

.
.
.
.

The optimal fit of any hoof boot requires regular trimming. We recommend at least all 6-8 weeks, depending on the hoof growth.

For optimal fit, measurements should be taken no longer than 2 weeks after your horses last trim.

If you experience any challenges to measure your horses hooves, visit youtube.com/swissgaloppers for further instructions.